
Tinkerine Face Shield V9.4: Manual

This face shield is used to protect the facial area from splashes, sprays, and spatters of bodily fluids. The current design is the result of continuous feedback from medical professionals. Not only is it balanced for functionality and comfort, it is also optimized for production speed and ease of assembly.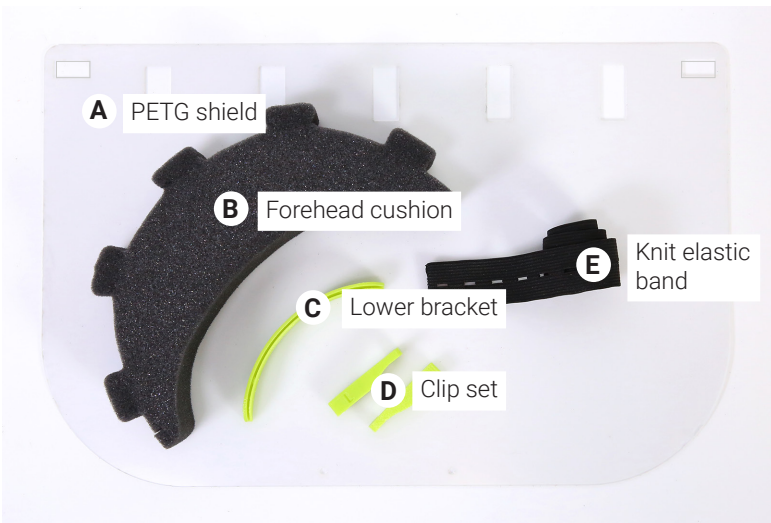

⚠ BEFORE ASSEMBLY

Make sure the protective films on both sides of the PETG shield are peeled off.

Step 1 - Attaching the clips + knit elastic band

- i. Take a clip and slide the tip into the top right corner hole on the PETG shield until the clip grabs onto the PETG shield.
- ii. Pop the centre of the clip into the PETG shield.
- iii. Repeat the above steps for the left corner hole.
- iv. Attach the knit elastic band onto the hooks on both sides. It's easier to hold the band perpendicular to the clip as shown in the picture.
- v. Check that the hooks are on the outside of the face shield.
- vi. Adjust the band for a snug but comfortable fit.

Step 2 - Attaching the cushion + lower bracket

- i. Take the forehead cushion and align its five extrusions to the five cutouts on the PETG shield.
- ii. Starting from one end, push all five foam extrusions through the cutouts.
- iii. Attach the lower bracket to the bottom of the PETG shield to help hold the shape.

Reusing the face shield

Only the PETG shield can be reused after sanitizing. Simply disassemble the shield in reverse order in which it was assembled. **Discard all other parts** as they cannot be reliably re-sterilized. Sanitize the PETG shield and reassemble the full face shield using fresh replacement parts.

Tinkerine Face shield is not sterile. Since the conditions and methods of assembly and use of Tinkerine Face Shield are beyond the manufacturer's control, the manufacturer expressly disclaim any and all liability arising from any assembly and use of Tinkerine Face Shield. NO WARRANTY OF FITNESS FOR ANY PARTICULAR PURPOSE, WARRANTY OF MERCHANTABILITY OR ANY OTHER WARRANTY, EXPRESSED OR IMPLIED, IS MADE CONCERNING TINKERINE FACE SHIELD. The user should thoroughly test any application before use. By assembling and using Tinkerine Face Shield, the user does so at their own risk and waives all claims against the manufacturer. The user releases the manufacturer from all liability, whether due to foreseeable or unforeseeable risks, relating to Tinkerine Face Shield. The user is solely responsible for their assembly and use of the Tinkerine Face Shield. The manufacturer is not responsible for errors or omissions relating to Tinkerine Face Shield. The manufacturer does not provide any medical advice.

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